

PUFFClicker™

Smart pMD Inhaler Tracker



**Compatible with pMD Inhalers:
Clenil Modulite, Salbutamol (Ventolin), Salbutamol (Ivax Salamol), Flixotide and Seretide.**

Activ8rlives' PUFFClicker can connect to:



Asthma+me App

Monthly Subscription
30-day free trial

Activ8rlives⁴

Health+Wellness App
Free of charge



www.activ8rlives.com

PUFFClicker connects via Bluetooth 4.0 and uploads data to Asthma+me App or Activ8rlives⁴ Health+Wellness App.
Works with: Bluetooth Smart Ready device (typically Bluetooth 4.0 or later). iPhone 5 or later, iPad 4th Generation or later, iPad Mini 2 or later, all iPad Air and iPad Pro models or iPod Touch 6th generation. Android devices running 5.0 or later. Amazon Kindle Fire devices running Amazon Fire OS 5.0 or later.

Contents


Your Activ8rlives PUFFClicker	4
Health and Safety Notices	5
Charging your device	6
How does the device work?	7
Compatible Inhalers	8
Assembling your PUFFClicker	9
Differences between Asthma ⁺ me & Activ8rlives ⁴ Health+Wellness Apps	10-11
Installing Asthma⁺me App &	
Accessing free 30 day trial of Asthma ⁺ me	12-13
Registration as Parent	14-15
About Your Child	15-17
Child's Medication	18
Daily, Weekly Surveys & Lessons	19
Care & Action Plan	20
Reports	21
Rewards	22
Installing Activ8rlives⁴ Health+Wellness App	23-24
Registration	25-26
Pairing your PUFFClicker	27-28
Putting into Pairing Mode	29
Navigating Screens	30
Taking an Inhaler Dose	31
Viewing Summary and Data	32
Adding Data Manually	33
Enabling Step Counting & Calculating Stride	34
Sending data to Cloud	35
Trouble Shooting	36
Rebooting Device	37
Specifications	38
Symbols & Warranty	39






We are a healthcare company. Our mission is to empower you to take care of your health and that of your family by self-care through self-monitoring.

From tracking levels of physical activity, weight, food intake, cardiovascular and lung function, to a new generation of at-home testing for long-term lung disease, we provide the devices and web-based tools to help you be active, eat well and stay healthy. Learn more at www.activ8rlives.com

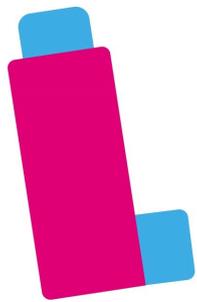

PUFFClickerTM

Smart pMD Inhaler Tracker

Your Activ8rlives PUFFClicker

Please read this manual carefully before you use your new device so that you do not injure yourself or others, or cause damage to your new device through improper use.

Intended Use


The **Activ8rlives PUFFClicker** is a counter for pressurised metered dose inhalers (pMDI) connected to **Asthma⁺me** or **Activ8rlives⁴ Health+Wellness Apps**, which allows you to count your doses, your steps, reminders to take medication and indicates that the inhaler has been shaken.

If in any doubt about your health, please consult your medical team first and follow their advice closely.

Your Activ8rlives PUFFClicker

The **Activ8rlives PUFFClicker** is made up of the **Module, Cap, Clip, USB Charging Cable** and prescribed Inhaler (not provided).

The **Activ8rlives PUFFClicker module** has a rechargeable battery, which cannot be removed. There are no user-serviceable parts and opening the **Module** case will void your warranty.

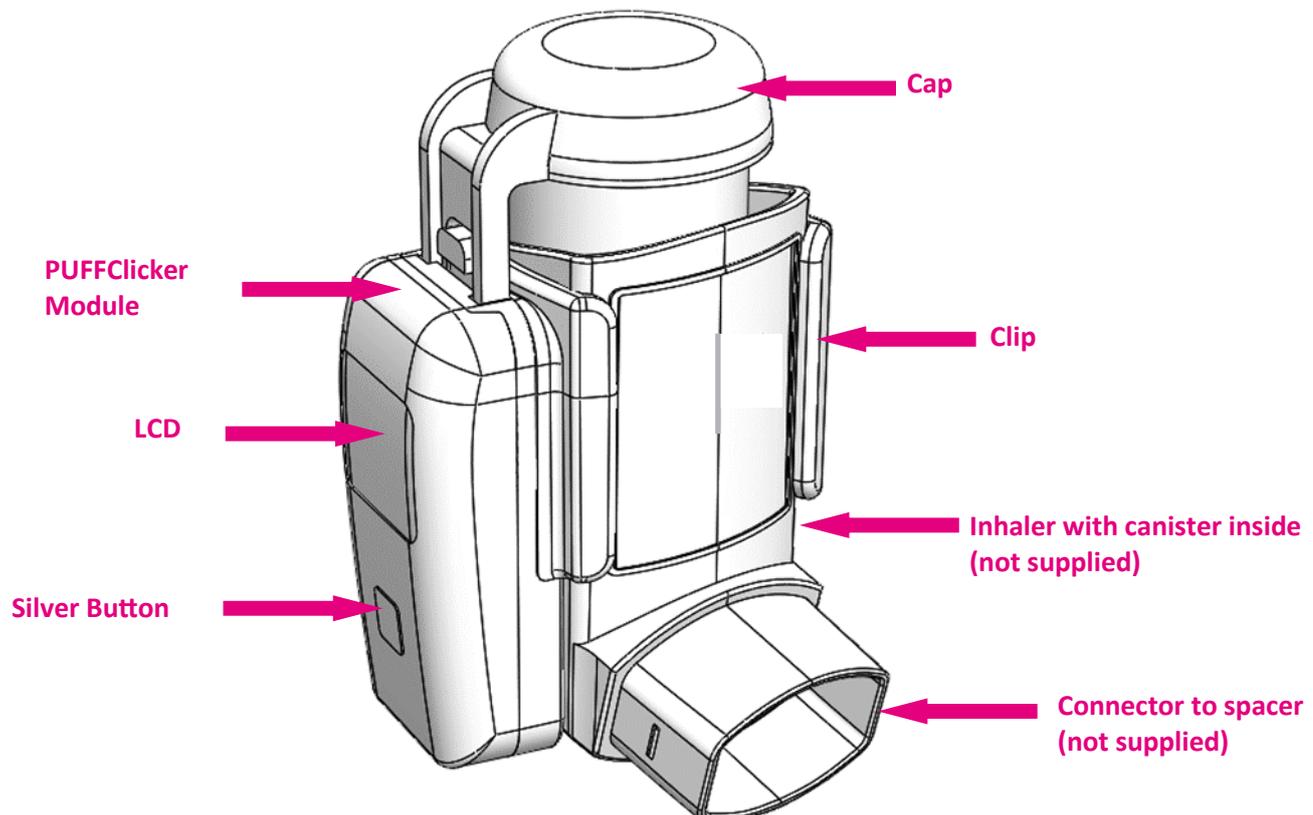

Health and Safety Notices

-  **WARNINGS:** The **Activ8rlives PUFFClicker** contains small internal components and magnets that may cause a choking hazard.
-  **WARNING:** Avoid using the **Activ8rlives PUFFClicker** in strong electrostatic and electromagnetic fields as these may cause the **Activ8rlives PUFFClicker** to reset. The **Activ8rlives PUFFClicker** will need to be reconnected to the **Asthma⁺me App** or **Activ8rlives⁴ Health+Wellness App** if this occurs.
-  **MEDICAL ALERT:** Asthma can be a life-threatening condition. Always take the medication as prescribed. If you experience any asthma-related symptoms of shortness of breath that does not get better after using your Reliever inhaler; trouble walking and talking because of shortness of breathe; blue lips or fingernails; severe wheezing; or a peak flow reading of less than 50% of your personal best, refer to your **Care Plan** and follow the instructions carefully. If you continue to have symptoms, consult your medical team. Do not rely on the **Activ8rlives PUFFClicker** to diagnose or manage your Asthma.
- This device is not a toy. Only recommended for children 5 years of age and over and only under adult supervision.
- Can be wiped clean with an antiseptic wipe and dried with a soft microfiber cloth. Don't use harsh cleaning products as these will cause damage to your **Activ8rlives PUFFClicker**. Don't allow sharp objects to scratch the device as this may damage it and make it more difficult to clean.
- The **Activ8rlives PUFFClicker Module** is not waterproof. Do not submerge it in water or put into a washing machine or dryer, as this will irreparably damage it.
- Do not allow your **Activ8rlives PUFFClicker** to be subjected to forces, shocks, dust, temperature changes or humidity over the range specified at the back of this User Manual. Harsh physical treatment may stop your **Activ8rlives PUFFClicker** from working properly. Don't expose your **Activ8rlives PUFFClicker** to direct sunlight for long periods of time.
- Don't remove the back cover of the **Module** and tamper with the internal components. If you do, you will damage the device. The **Activ8rlives PUFFClicker** also contains small internal components, which present a serious choking hazard for small children. Only use under adult supervision.
-  Do not rely solely on the **Reminder** capabilities of the **Activ8rlives PUFFClicker**, **Asthma⁺me App** or **Activ8rlives⁴ Health+Wellness App**, to take your prescribed medication as directed by your Medical Team.
- The **Activ8rlives PUFFClicker Module** contains a rechargeable lithium battery. It takes about 120 minutes to charge and with typical use, will last 7 days before recharging is required. Only use the **Magnetic USB Charging Cable** supplied. Avoid rapid chargers over 1Amp as these decrease the lifetime of your rechargeable battery. The typical lifetime of this products rechargeable battery is 200-300 charges. If you are planning to store the device for a long period of time, charge it first before storage. When disposing of the **Module**, ensure that it is collected separately for special treatment as it contains a lithium battery that should not be disposed of as normal household waste.
-  If you experience any serious incident that occurs in relation to this device, please report this to Aseptika Limited (**Activ8rlives**) and the competent authority of the Member State in which you are based.
-  The technical specifications for this product and the contents of this User Manual are subject to change without notice by the manufacturer.

Customer Support 7-days-a-week, between 09:00—18:00
UK +44 (0)1480 352 821 or support@aseptika.com


Charging Your Activ8rlives PUFFClicker

Your **Activ8rlives PUFFClicker** has a rechargeable lithium battery integrated into the **Module**. It takes about **120 minutes** to fully charge with the **Magnetic USB Charging Cable** supplied. A full charge gives 7 days of life.

You can charge your **Activ8rlives PUFFClicker** by connecting the magnetic end of the **USB Charging Cable** (with the two small round magnets) to the bottom of the **Activ8rlives PUFFClicker Module** at the corresponding magnets. The other end of the **USB Charging Cable** (flat end) insert into a PC, laptop or power strip. Only use the **USB Charging Cable** supplied as others are not compatible.

If the **Activ8rlives PUFFClicker** is running low on power, the **LOW BATTERY**  warning symbol will appear on the **LCD screen**. When fully charged the symbol will appear as .

Note: Do not use rapid chargers or chargers outputting more than 1Amp of current as this will damage the battery. Only use Laptops, PCs or chargers with 0.5Amp or 1Amp.

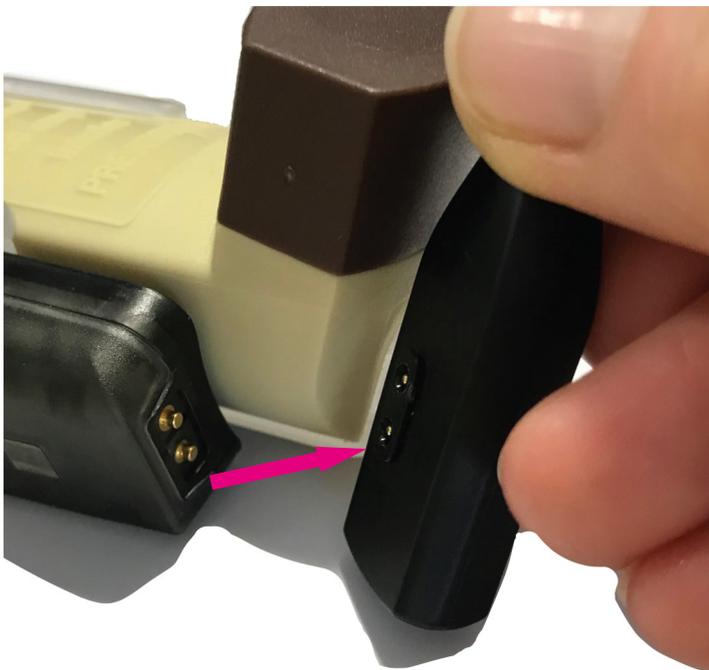

Attach the pins at the bottom of the **Activ8rlives PUFFClicker** to the corresponding magnetic pins on the **USB Charging Cable**, making sure that both pins are touching.



Slot the the **USB Charging Cable** (flat end) into a PC, laptop or power strip. **Only use Laptops, PCs or chargers with 0.5Amp or 1Amp.**

How Does the Activ8rlives PUFFClicker Work?

The **Activ8rlives PUFFClicker** has a built-in sensor and counter that enables it to count the number of puffs you take of your Asthma pMD inhaler when you press down on the **Cap** at the top of the device.

It also has a 3-axis accelerometer chip and a microprocessor, which detects how you move (accelerate) in three dimensions and this allows it to count how many steps you take each day if you carry it with you (ideally carry your blue Reliever with you everywhere). It also is used to calculate if you have shaken your inhaler, before you use it, to maximise the inhaler drug delivered to your lungs.



Note: Do not tamper or attempt to open the module in which the rechargeable battery is housed. Your warranty will be invalidated if you do. Substances contained within the battery may damage the environment or human health if handled and thus there is a risk of electrical shock or burns. When disposing of this product and its integrated battery, ensure that it is collected separately for correct disposal and not as normal household waste.

Which Inhalers are Compatible with the PUFFClicker?

The **Activ8rlives PUFFClicker** is a smart dose counter to be used with pressurised metered-dose inhalers (pMDI) and is available in 4 different sizes so that they can be used with these different inhalers listed in the table below.

The **Activ8rlives PUFFClicker** is clipped over your inhaler and is transparent in colour so you can easily see which inhaler you are about to use.

Preventers	Relievers
<p>Clenil Modulite</p> <p>Clenil® Modulite® Chiesi Limited</p> 	<p>Salbutamol Ventolin</p> <p>Ventolin® GlaxoSmithKline</p> 
<p>Flixotide / Seretide</p> <p>Flixotide® GlaxoSmithKline / Seretide® GlaxoSmithKline</p> 	<p>Salbutamol Ivax Salamol</p> <p>Ivax Salamol® Teva Pharmaceuticals</p> 

Assembling Your PUFFClicker


Select the **Activ8lives PUFFClicker** that match your inhaler(s) that you use everyday or occasionally, which then needs to be assembled with your prescribed inhaler(s) (not included).

Your **Activ8lives PUFFClicker** consists of: **Module**, **Cap** and **Clip**, which fits together easily by following the instructions below. When fitting the parts together, you will hear a **CLICK** when each part is in its correct position.

Step 1– Remove Inhaler from packaging




Step 2– Attach Clip to Inhaler > CLICK

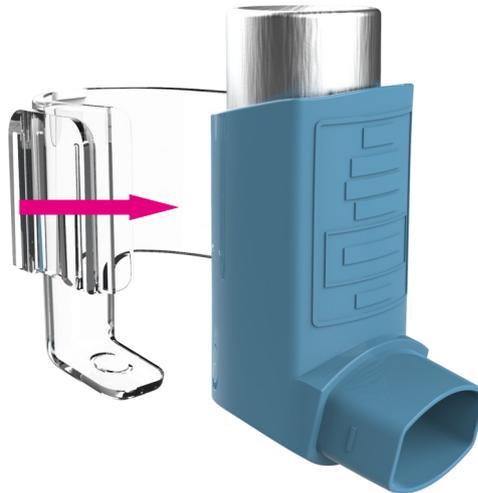

Step 3– Attach Module to Clip > CLICK



Step 4– Attach Cap to Module, press in button to slide in Cap > CLICK

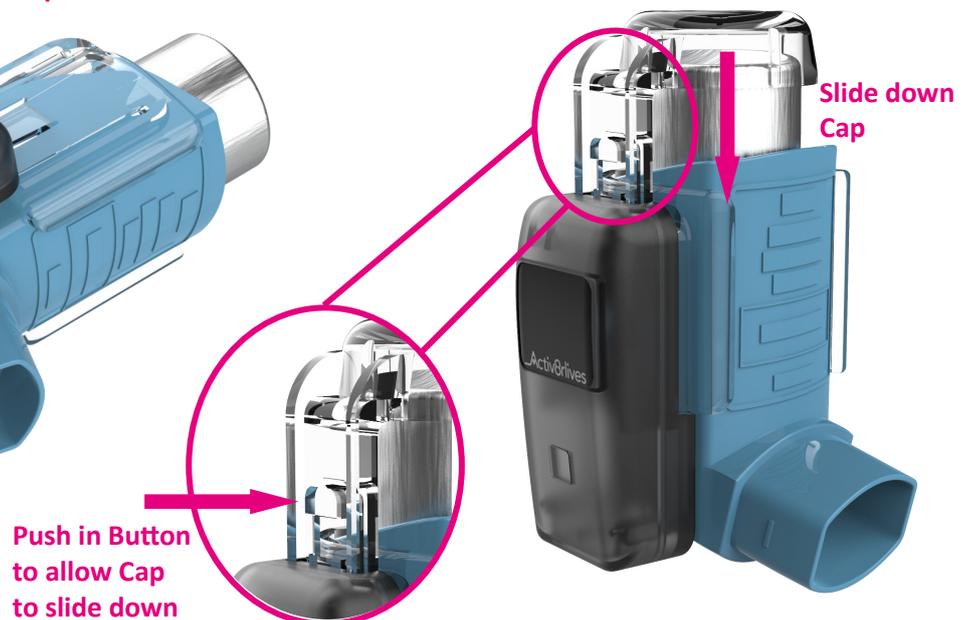

Which Activ8rlives App Will Suit the Needs of your Asthma?

Feature	Asthma ⁺ me 	Activ8rlives ⁴ Health+Wellness App 
Cost	Monthly subscription 30-day free trial	Free of charge
Parent / Child Registrations	✓	
Count your Inhaler usage	✓	✓
List your Triggers	✓	
Add your Medication	✓	✓
Reminder to do Daily monitoring tasks	✓	
Set-up Care and Action Plan and Print	✓	
Daily Symptom Surveys	✓	
Weekly Symptom Surveys	✓	
Asthma Lessons	✓	
Emergency Contact Details for Parent/Carer	✓	
Set-up Rewards to increase adherence to Care Plan	✓	
Connect to your Hospital account (where supported)	✓	
Children can customise the look of their account	✓	
Generate Reports of Asthma data and print	✓	
Adding Data Manually	✓	✓
Weather and Pollution details in your location	✓	
Connect other Activ8rlives devices	✓	✓



PUFFClicker™

Smart pMD Inhaler Tracker



The **Activ8rlives** PUFFClicker connects via Bluetooth to either:

Asthma⁺me App
(Monthly Subscription)
30-day free trial



Activ8rlives⁴ Health+Wellness App
(Free of charge)





Let's get started with the Set-up and Connect to the Asthma⁺me App

The **Activ8rlives PUFFClicker** is programmed by and uploads information directly to and from your **Activ8rlives** account each time your Smartphone or Tablet (not included) is connected to it wirelessly by Bluetooth 4.0.

This guide will show you how to:



Install the **Asthma⁺me App** to your iOS or Android Smartphone or Tablet;

Register as a Parent and your Child, or just for Yourself;

Settings and features to personalise your **Asthma⁺me** experience.

If you follow these directions sequentially, it will take you about 10-15 minutes to learn how to count your inhaler doses, activity and connect to the **Asthma⁺me App**.

Skip to Page 23 if you wish to use the free of charge Activ8rlives⁴ Health+Wellness App

Intended Use for the Asthma⁺me App :

Asthma⁺me App is a mobile application intended for use in the home to assist people to receive and review information from connected medical and non-medical devices, track medication use and record other events such as asthma attacks, for effective health self-management. The user also can share the data via sharing functions.

The is classified as a Class 1 medical device under MEDDEV 2. 4/1 Rev. 9 June 2010 as per the requirements set out in Annex IX of Directive 93/42/EEC as amended 2007/47/EC.

Health and Safety Notices for the Asthma⁺me App:

- ⚠ **Asthma⁺me App** is not intended to monitor vital physiological parameters where variation could result in immediate danger.
- ⚠ Asthma can be a life-threatening condition. Always take the medication as prescribed. If you experience any asthma-related symptoms of shortness of breath that does not get better after using your Reliever inhaler; trouble walking and talking because of shortness of breath; blue lips or fingernails; severe wheezing; or a peak flow reading of less than 50% of your personal best, refer to your **Care Plan** and follow the instructions carefully. If you continue to have symptoms, consult your medical team.
- ⚠ Do not rely on the **Asthma⁺me App** to diagnose or treat your Asthma.
- ⚠ Only recommended for children 5 years of age and over and only under adult supervision.
- ⚠ Do not rely solely on the **Reminder** capabilities of the **Asthma⁺me App** to take your prescribed medication as directed by your medical team.

If you experience any serious incident that occurs in relation to the **Asthma⁺me App**, please report this to Aseptika Limited (**Activ8rlives**) and the competent authority of the Member State in which you are based. Contact details are on the back of this User Manual.



Installing the Asthma⁺me App on Your Smartphone or Tablet (Bluetooth 4.0 Enabled Devices are required to use the PUFFClicker and Asthma⁺me App requires a Monthly Subscription).

Download the Asthma⁺me App from the App Store appropriate to your device. Search for Asthma⁺me in the search bar of your App Store or scan the QR code below and it will take you to the appropriate App Store for your device.



Asthma+me



Apple devices running iOS 10 or above that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later) - iPhone 5 or later, iPad 4th generation or later, iPad Mini 2 or later, all iPad Air and iPad Pro models or iPod touch 6th generation.



Android devices running Android 5.0 or later that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later).



Amazon Kindle Fire devices running Amazon Fire OS 5.0 or later that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later) - Kindle Fire (5th generation or later), Kindle Fire HD and Kindle Fire HDX (4th generation or later).

You can use the Asthma⁺me App for 30-days free trial to see if it suits the needs of your asthma.

IMPORTANT NOTE: Please adjust the screen lock function on your Smartphone or Tablet to be longer than 2 minutes to allow the Activ8lives PUFFClicker time to sync with your device.

Short times of less than 2 minutes cause frequent loss of connectivity.



Help Videos in App, follow this ?



Website www.Activ8lives.com

YouTube <http://bit.ly/Activ8lives>

Still Need Help

We are here to help you 7-days-a-week, so please call us to help you set-up your device on UK +44 (0)1480 352821.



Menu

Asthma+me

Email Address

Password

Login

Health and Safety Notices

Read Notices

I have read and understood the Health and Safety Notices

Forgotten Password?

Reset password

Not a Member?

Register an account

Menu Register ?

How would you like to use Asthma+me?

Just for myself

I am over 13 years old and I am monitoring my own asthma.

I am a parent

I will be monitoring the asthma of my child or children.

Important:

Asthma+me requires a paid subscription to unlock functionality. A free trial is available on completion of the registration process. You may cancel your subscription at any time.

Why Register as a Parent?

Use this option if your child is 12 years and under, or not yet ready to manage their own asthma.

These instructions are for a **Parent** (we call them the **Carer**) that manages their child(ren)'s asthma and require access to their child's data. Your child can have their own smart devices or you can share your device with them.

Please follow the instructions provided on the **Asthma+me App** screen if you're **Registering** an account as a **Parent** for your child.

Register as Parent (Carer)

Once you have installed the **Asthma+me App**, open the App making sure your Smartphone or Tablet is connected to the internet with **Bluetooth switched on**.

If you have already **Registered**, you can **Login** to your **Activ8lives** account. Click the **Login** icon and enter your **Username** and **Password**.

If you have not previously **Registered** for your **Activ8lives** account, then click the **Register an Account** button on the App screen.

Remember your **Username** (email) and **Password** details associated with your account and keep these safe. You use the same **Username** and **Password** to access your account via the web: www.activ8lives.com.

Note: Your contact details, including your telephone number will be used in the **Emergency Contact Details** section of the child's **Care and Action Plan**.

< Menu Register ?

I am registering an account for my child...

Use this option if:

- ✓ You require access to the data generated by your child
- ✓ You and your child have your own smart device ?
- ✓ Or, you and your child will be sharing one smart device ?

Two accounts will be created during the registration process: one for you and one for your child.

Back Next

< Menu Register ?

Carer Registration

This information will be used to fill in emergency contact details in your child's asthma action plan.

Relationship to caree

Select from the list

Phone number ?

I do not wish to enter my phone number

Back Next

< Menu Register ?

Carer Registration

This information will be used to fill in emergency contact details in your child's asthma action plan.

Address line 1

Address line 2

Address line 3 (optional)



Menu Register ?

Carer Registration

The PIN will be used to prevent your child from accessing important areas of the Asthma+me app that only the parent should use

PIN ?

PIN confirm

Skip

Back Next

Completing Parent Registration

Entering a **PIN number** is optional but it is recommended for younger children. The **PIN** will prevent your child from accessing and changing important information about their Asthma and its care.

Kaya Menu

Family

Dashboard

Reports

Medication

Rewards Management

Learn

Care Plan

Charts

Add Data

Devices

Menu

and they can continue using Asthma+me as they wish.

Do not show this message again

Kaya

Kaya

Current Active User

Carees

You do not yet have any Carees

Tap the button below to add your first Caree

Add a new Caree

Add a new Caree to manage their condition via Asthma+me.

Add Caree

Adding Your Child(ren) (Caree)

After **Login** as a parent, open the **Menu** and navigate to **Family**.

The top account is the Parent account, as you act as the **Carer**. Touch **Add First Caree** (that's the name we call a Child or Young Person) to start the **Registration** process for your child.

Please complete the **Registration** process with your child as they will be able to choose different **Avatars** and **Themes** for their account.

Make sure the **Carer** account is active. Touch the **Add Caree** button to proceed.

Top account Parent/Carer Active

In the second screen the top account is your **Carer** account. Click **Add a Caree** to start the **Registration** process for your child.

Menu Caree Registration ?

Add Caree Remove Caree

Caree Registration

All data entered in this form is private to you, your family and any clinician you choose to share it with, unless indicated with PUBLIC

My first name is...

My surname is...

I like to be called... PUBLIC

Please complete the **Registration** process with your child as they will be able to choose different avatars and themes for their account.

Note: Only cells containing a **Public** tag will be visible to other users on **Activ8rlives** Apps if the child subscribes to a Support Group, i.e. in the examples on the left-hand side **I Prefer to be called** and highlighted **Avatar** will be public, but not the full name of the **Caree**.

Note: You must set-up a new account for each child that you care for.



Menu Caree Registration ?

Add Caree Remove Caree

Caree Registration

All data entered in this form is private to you, your family and any clinician you choose to share it with, unless indicated with PUBLIC

My first name is...

My surname is...

I like to be called... PUBLIC

About Your Child (Caree)

We need to add a little more information about your child (Caree) so their **Care Plan** will identify them. What is their name and what would they like to be called — **I like to be called?**

Note: Only cells containing a **Public** tag will be visible to other users on **Activ8lives** Apps if the child subscribes to a Support Group, i.e. in the examples on the left-hand side **I Prefer to be called** and highlighted **Avatar** will be public, but not the full name of the **Caree**.

Avatars and **Themes** will personalise your child's **Asthma⁺me App** interface and they will be able to change this in **Settings** at any time.

Choose the **Theme** from the pull down menu and touch **Next**.

Menu Add a Caree ?

My avatar is... PUBLIC

The image displays a grid of 12 app theme preview cards. Each card shows a different color scheme and layout for the app's interface. The themes are: Bulldog Theme (red), Festive Theme (green), Spicy Hot Theme (orange), Cucumber Theme (yellow), Ice Blue Theme (blue), Autumn Gold (gold), and Activ8lives Default Theme (light blue). Each card includes a date '29 May 2018', 'Inhaler Use Summary', 'Preventer Puffs', 'Reliever Puffs', 'Puff Schedule', 'Reliever', and 'My PUFFClickers' sections.



About Your Child's Asthma

Gender and date of birth are used to calculate health parameters such as your child's target peak flow.

Your child's peak flow will be automatically calculated based on their parameters for age, gender and height provided upon **Registration**. You can input their own targets if they have been given different values by their doctor.

Tick all of the triggers that apply to your child (Caree). These settings will be used to populate your child's **Care & Action Plan** and to make the **Asthma+me** experience relevant to them.

During the **Registration** process you can select any **Activ8lives** devices that you have. Once you have completed your **Registration** you will automatically be directed to your **Dashboard** and these devices will now show up on your **Dashboard** so that you can quickly take a reading by touching the image of the device and it will start the upload process.

Remember your **Username** (email) and **Password** details associated with your account and keep these safe. You use the same **Username** and **Password** to access your account via the web: www.activ8lives.com.

Registration for 13 Years—Adults

Anyone over the age of 13 years and that doesn't want to share their data with a Parent/Carer may set-up their own account for themselves.

Please follow the instructions provided on the **Asthma+me App** screen if you're **Registering** an account **Just for myself**.

Note: Details for the **Emergency Contact Details** that are inserted into your **Care & Action Plan** will need to be that of a **Parent /Carer /Partner** so that in an emergency someone else other than you can be contacted.

Dashboard for Carer Viewing Child's data

Dashboard for Child/Caree



Set-up, Manage & Track Your Medication(s)

These pages will help you set-up, manage and track your medications.

Open the **Menu** and navigate to **Medication**.









Please choose the type of medication: Preventer, Reliever, Steroid, LTRA, Antibiotic, Bronchodilator, Decongestant, Antihistamine or Other.

Preventers, Relievers and **LTRAs** have pre-configured options to help you select the correct medication. Refer to the packaging to enter the correct name, strength and quantity of doses.

Add a **Reminder** for each time you have to take this medication, or you can toggle the button to take the medication when necessary. Add the number of doses you have to take each time when the reminder appears.

To **Change the Batch** of a recorded **Medication** (**Asthma+me App** navigate to **Menu > Family** and make the Carer active) or (**Activ8lives⁴ Health+Wellness App** select **Medication**) select the **Medication** you wish to edit, touch **Track a dose**, then **Close batch**. Follow the instructions above to **Add batch**.

Note: See Page 8 for compatible pMDIs for the **Activ8lives PUFFClicker**.

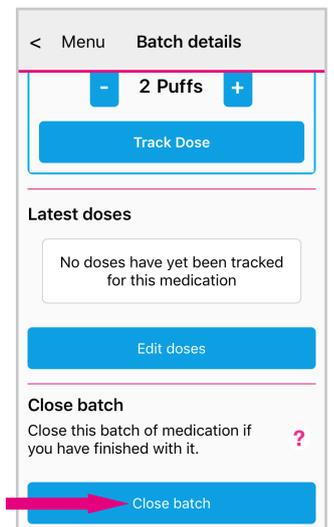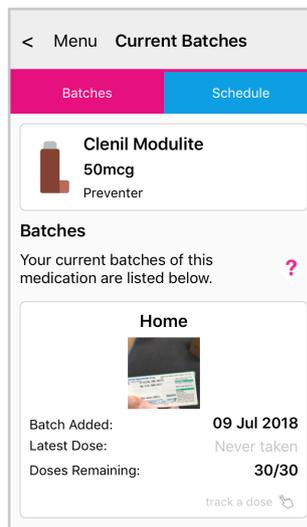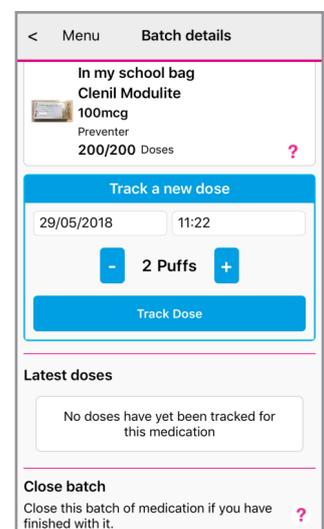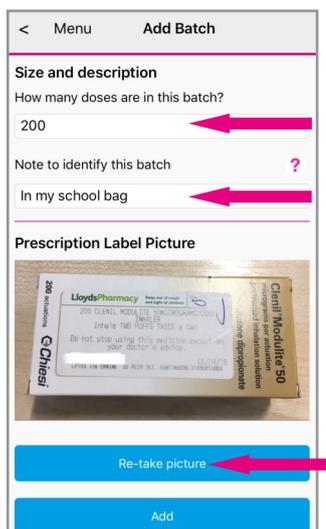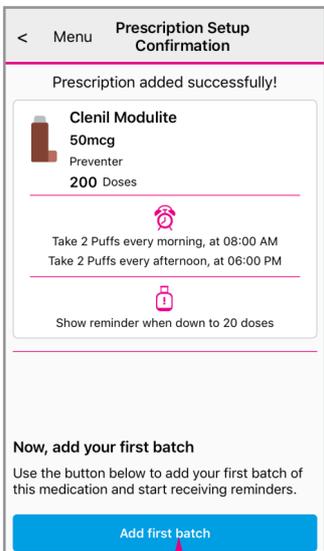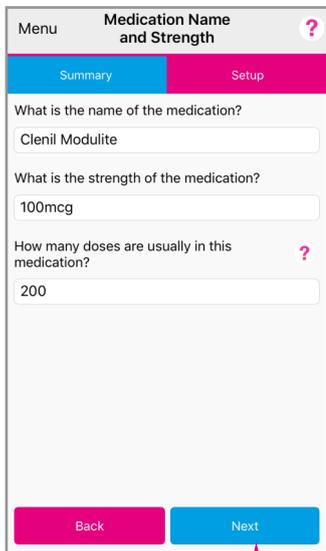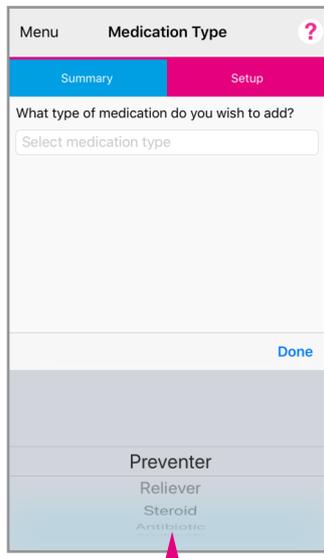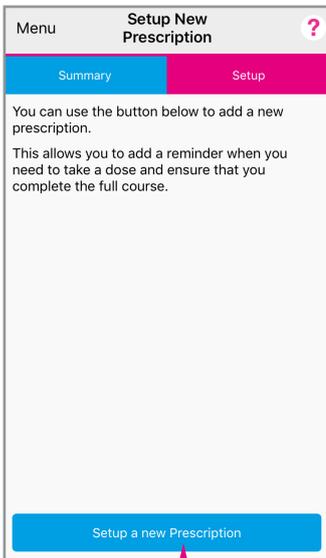



Daily Asthma Dairy

You can take a **Daily Asthma Dairy** to track how your Asthma is affecting your daily life.







Weekly Asthma Survey

From your **Dashboard**, touch the tile for your **Weekly Survey** and this helps to see how your Asthma symptoms have been over the course of a week.

Asthma Lessons


A series of short multimedia **Lessons** are available for you to view and work through.

A **Quiz** at the end of each **Lesson** may have one correct answer or can have multiple answers. Correct answers will light-up in **Green** and you will then be allowed to finish the **Lesson**. Incorrect answers will light-up in **Red**, and you will be asked to try again.

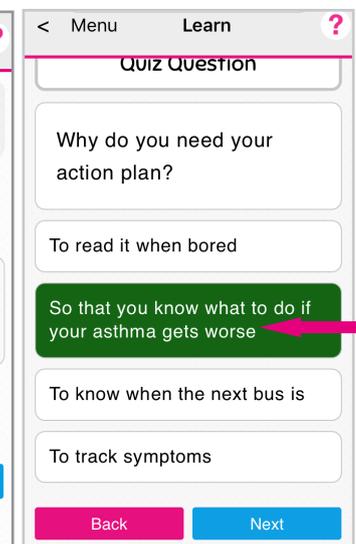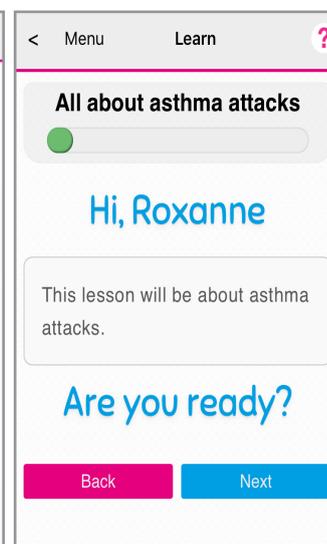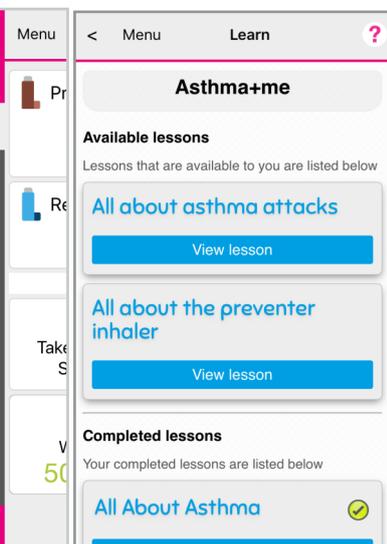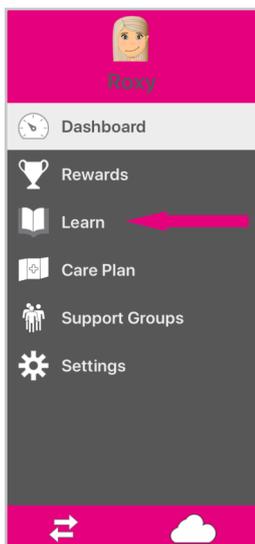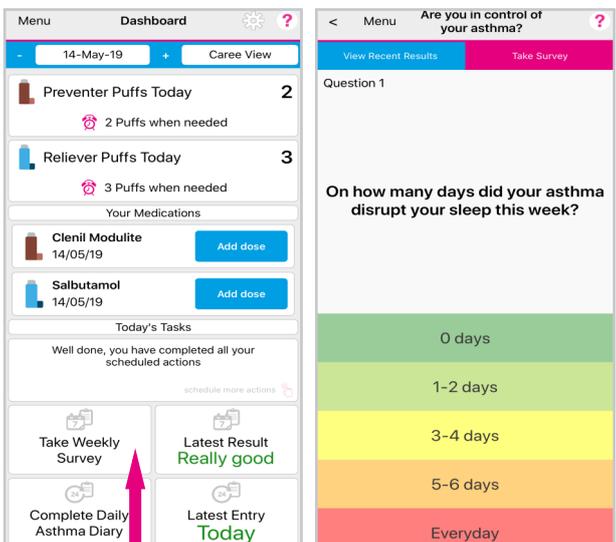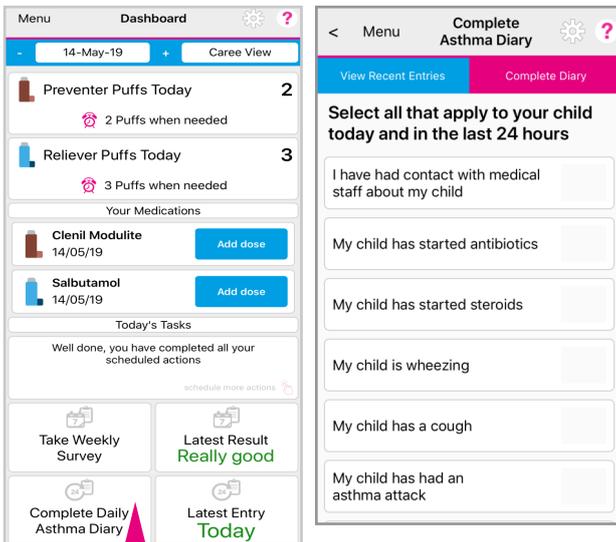



↓ Access via Menu, select Care Plan

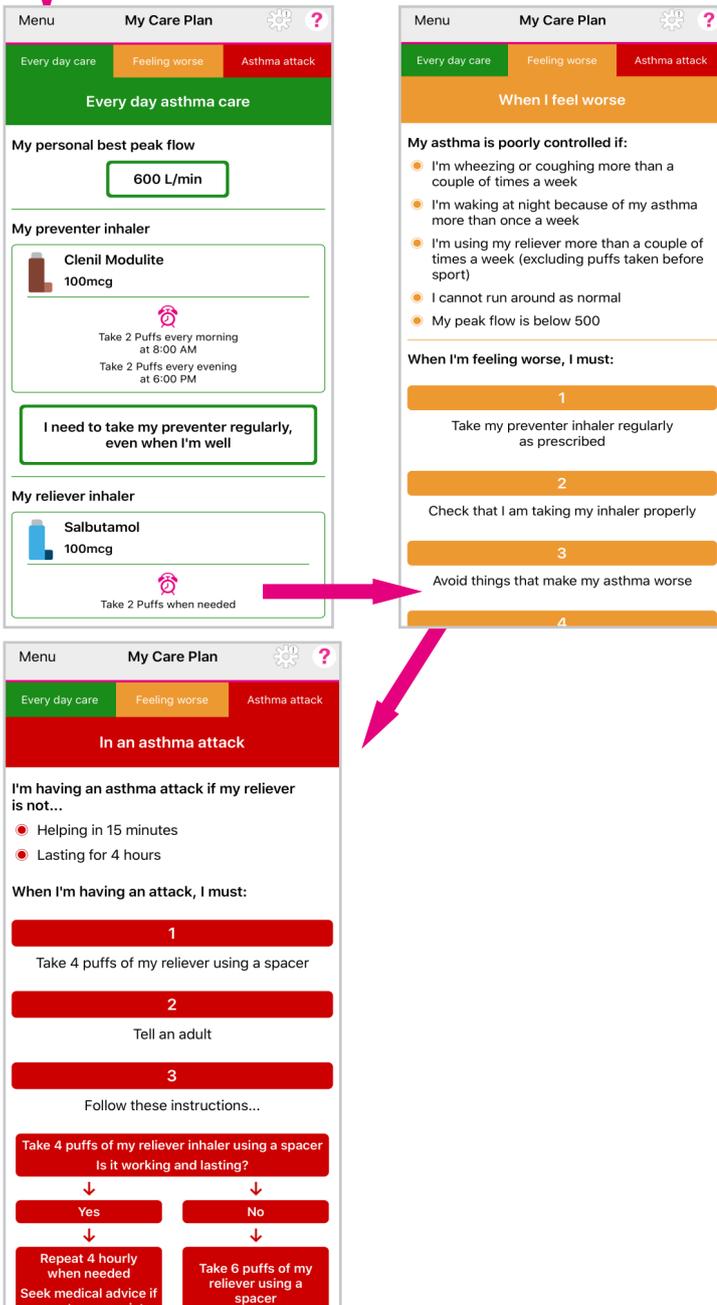

Care and Action Plans

Everyday Asthma Care

The first part of your **Care Plan** is what you need to do everyday to stay well - **Everyday Asthma Care**. From this you and anyone who provides medical care for you will know your personal best peak flow, what is the name and strength of your inhalers, how many puffs of preventer you need to take and when to take your reliever based on your **Care Plan**. It will also inform you and anyone who provides medical care for you of what other medication you are taking. It will also list your asthma triggers and your next Asthma Review.

When I Feel Worse

The second part of your **Care Plan** will let you and anyone who provides medical care for you recognise signs that you're **feeling worse** and steps you can take straight away to get your asthma back under control and help you feel better.


You should follow the instructions carefully to get your Asthma back under control.

In an Asthma Attack

The third part of your **Care Plan** will let you and anyone who provides medical care for you know what to do in case of an **Asthma Attack**. This section contains the contact details of your **Emergency Contact** (usually a parent/carer/partner), signs showing you're **In an asthma attack** and what emergency actions to take to help you.

You should show this section to an appropriate adult if you think that you are having an asthma attack (such as teacher, scout leader, friend's parent, partner, work colleague).

The **Care & Action Plan** can be downloaded as a PDF file that you can send via Email and print out copies.

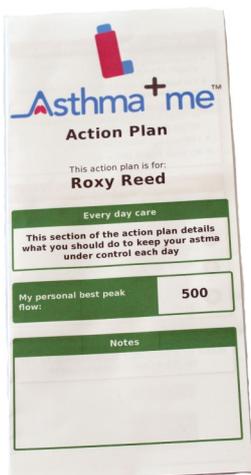



Generate Asthma Reports


To set-up automatic generation of **Reports** for your **Child**, open the **Menu** and navigate to the **Family** page, and make sure your child's profile is active, by touching the toggle button to **Green**.


From the **Asthma+me App** you can schedule **Reports** to be generated into a PDF format about yourself or your child. The **Report** is delivered so you may share this with your Medical Team, Asthma Nurse, Hospital Clinician, GP by email or just show this at your next Asthma Review.

To generate a **Report** select **Reports** from the **Menu**. Touch the **Schedule** tab. You can generate a one-off **Report**.

You can also schedule a regular **Report** using the **Schedule Report** section. The **Report** will be generated overnight on the selected day. You will be emailed when the **Report** is ready to be viewed.

Once your **Carer** has received an email that your **Report** has been generated, select **Reports** from the **Menu** and you will see the **Report** waiting for you to view. The **Report** will open as a PDF file on your Smartphone or Tablet. You can print the **Report** using your Smartphone or Tablet print options.

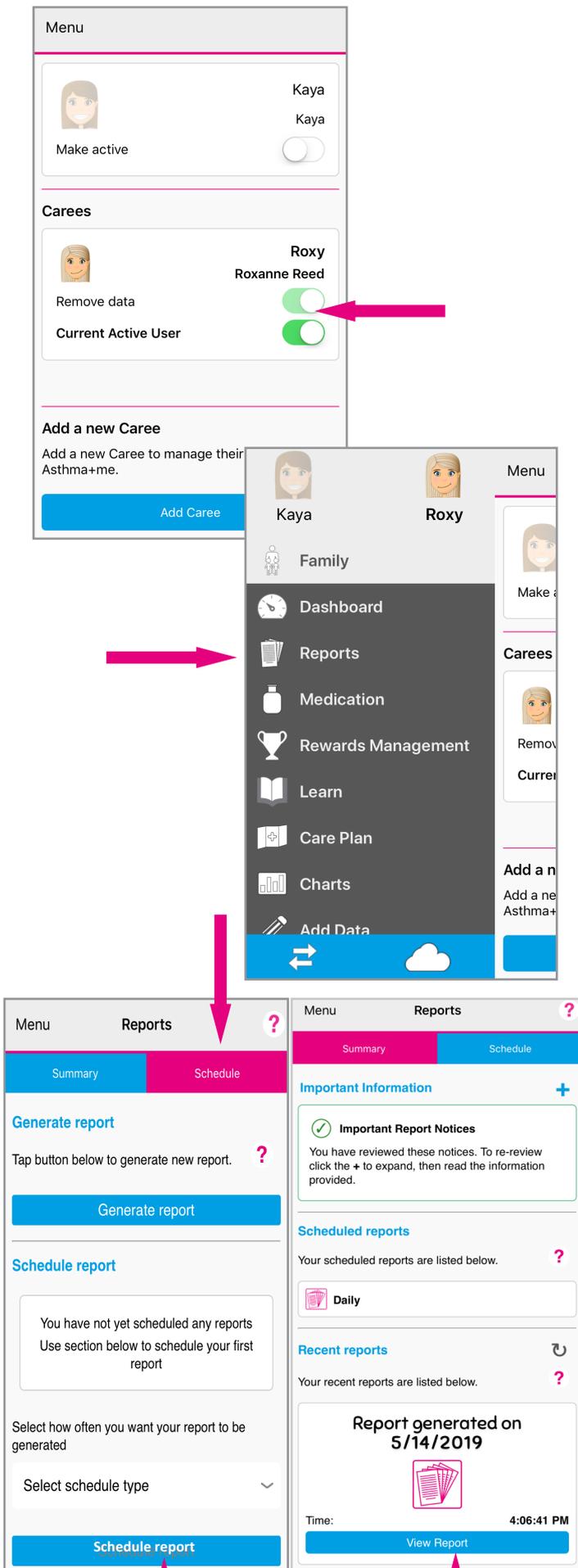

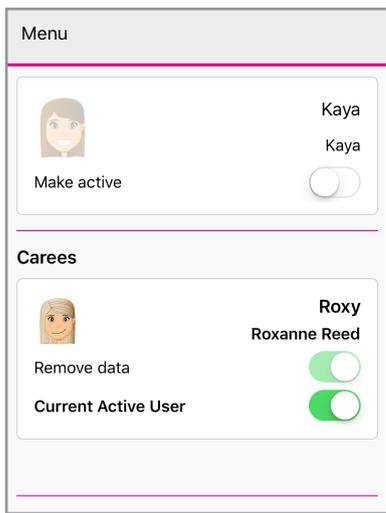

Rewards

The **Asthma⁺me App** has been designed to incorporate **Rewards** for your Child (Caree) which helps to encourage them to adhere to taking their medication as prescribed and to engage in learning more about their Asthma.

Involve your child in setting an achievable target to earn their **Rewards**.



From the **Menu** select **Rewards Management** and click on the **Setup** tab. Follow the instructions on the **Asthma⁺me App**, by completing the Name of the Reward, Image, Points Target and tick the behaviours that will incentivise them to complete the Daily and Weekly tasks.

You can use the Calculator provided in the **Asthma⁺me App** to work out how long it will take them to earn the **Reward** based on the behaviours you wish to encourage.

Only Parents/Carers can set-up **Rewards**, so make sure the Carees account is active.

Children can view their progress by accessing the **Rewards** page from the **Menu**.

Make this fun for them and reward them for taking care of their Asthma and learning more about how to keep well!

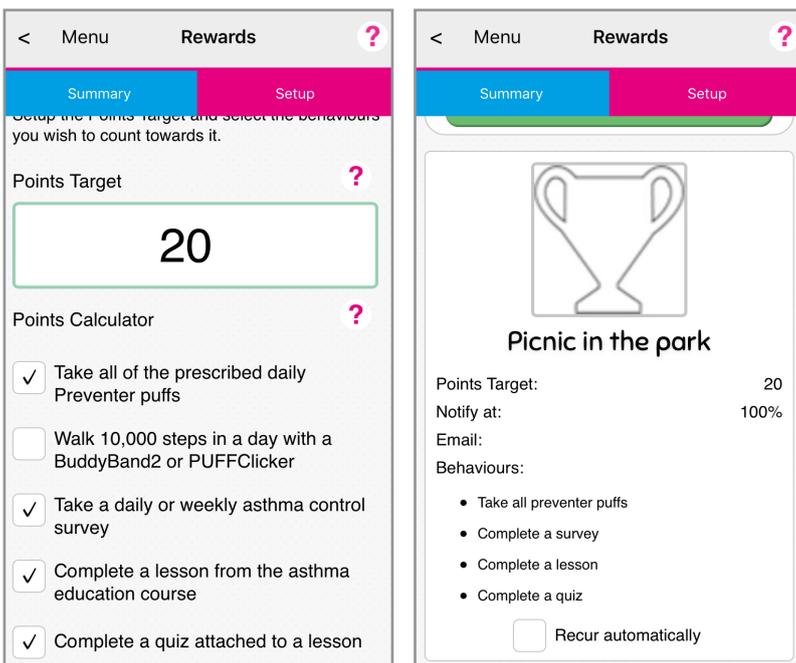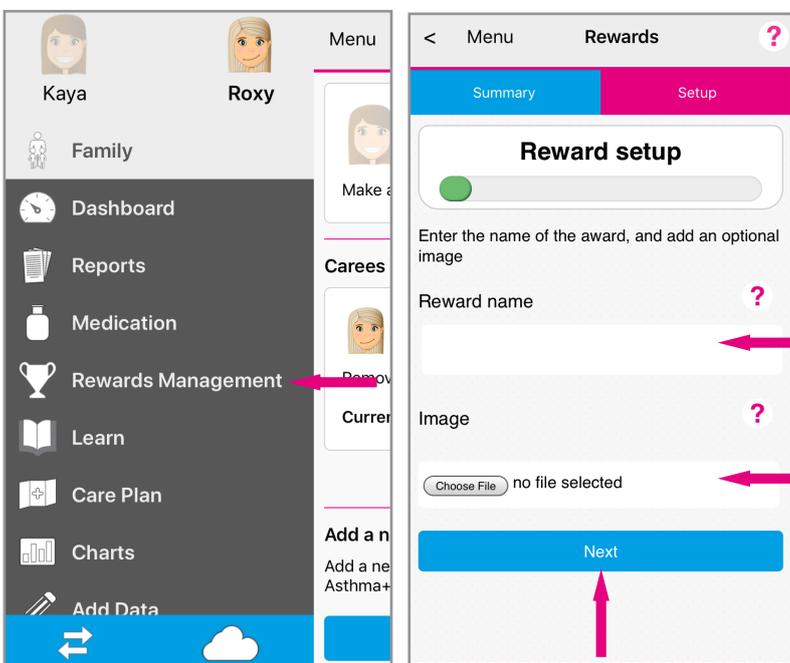



Let's get started with the Set-up and Connect to the Activ8lives⁴ Health+Wellness App (Free App no subscription required)

The **Activ8lives PUFFClicker** is programmed by and uploads information directly to and from your **Activ8lives** account via the **Activ8lives⁴ Health +Wellness App** each time your Smartphone or Tablet (not included) is connected to it wirelessly by Bluetooth 4.0.

This guide will show you how to:



Install the **Activ8lives⁴ Health +Wellness App** to your iOS or Android Smartphone or Tablet; and

Register for an **Activ8lives** Account.

If you follow these directions sequentially, it will take you about 10-15 minutes to learn how to count your inhaler doses, activity and connect to the **Activ8lives⁴ Health +Wellness App**.

Skip to Page 12 if you wish to use the 30-day free subscription of Asthma⁺me App

Intended Use for **Activ8lives⁴ Health+Wellness App**:

Activ8lives⁴ Health+Wellness App is an application software for mobile device intended for use in the home to assist people to receive and review information from connected medical and non-medical devices, track medication use, for effective health self-management. The user also can share the data via sharing functions. The **Activ8lives⁴ Health+Wellness App** is intended for use with adult and pediatric patients over five years of age. **Activ8lives⁴ Health+Wellness App** is a Class 1 Medical Device.

Health and Safety Notices:

Activ8lives⁴ Health+Wellness App is not intended to monitor vital physiological parameters where variation could result in immediate danger.

Long-term health conditions (LTHC) can be life-threatening. Always take medication as prescribed. If you experience any LTHC-related symptoms, refer to your **Care Plan** and follow the instructions carefully. If you continue to have symptoms, consult your medical team.

⚠ Do not rely on the **Activ8lives⁴ Health+Wellness App** to diagnose or treat any long-term health condition. Only recommended for children 5 years of age and over and only under adult supervision.

⚠ Do not rely solely on the Reminder capabilities of the **Activ8lives⁴ Health+Wellness App** to take your prescribed medication as directed by your medical team.



Installing the Activ8lives⁴ Health+Wellness App on Your Smartphone or Tablet (Bluetooth 4.0 Enabled Devices are required to use the Activ8lives PUFFClicker)

Download the **Activ8lives⁴ Health+Wellness App** from the App Store appropriate to your device. Search for **Activ8lives** in the search bar of your App Store or scan the QR code below and it will take you to the appropriate App Store for your device.



Activ8lives



Apple devices running iOS 10 or above that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later) - iPhone 5 or later, iPad 4th generation or later, iPad Mini 2 or later, all iPad Air and iPad Pro models or iPod touch 6th generation.



Android devices running Android 5.0 or later that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later).



Amazon Kindle Fire devices running Amazon Fire OS 5.0 or later that are Bluetooth Smart Ready (typically Bluetooth 4.0 or later) - Kindle Fire (5th generation or later), Kindle Fire HD and Kindle Fire HDX (4th generation or later).

The **Activ8lives⁴ Health +Wellness App** is free to use for the needs of your asthma.

IMPORTANT NOTE: Please adjust the screen lock function on your Smartphone or Tablet to be longer than 2 minutes to allow the **Activ8lives PUFFClicker** time to sync with your device.

Short times of less than 2 minutes cause frequent loss of connectivity.



Help Videos in App, follow this ?

Still Need Help



Website www.Activ8lives.com

We are here to help you 7-days-a-week, so please call us to help you set-up your device on UK +44 (0)1480 352821.

YouTube <http://bit.ly/Activ8lives>



Menu

Activ8lives

Email Address

Password

Login

Health and Safety Notices

Read Notices

I have read and understood the Health and Safety Notices

Not a Member?

It is free to join Activ8lives!

Register

Take a Tour

Register for Activ8lives Account

Once you have installed the **Activ8lives⁴ Health+Wellness App**, open the App making sure your Smartphone or Tablet is connected to the internet with **Bluetooth switched on**.

If you have already Registered, you can **Login** to your **Activ8lives** account. Click **Login** and enter your **Username** and **Password**. Skip to **Page 27** as you have already registered.

If you have not previously Registered for your **Activ8lives** account, then click **Register** on the App screen.

< Menu Registration ?

Units and Measurements

We use your height and weight to measure your well as a few other parameters. Let us know which you prefer so that we show you your data in a way that makes sense to you

Units

Metric

Weight Units

Kg

Weight

Height Units

Cm

Height

Menu

and they can continue using Asthma+me as they wish.

Do not show this message again

Kaya

Kaya

Current Active User

Carees

You do not yet have any Carees

Tap the button below to add your first Caree

Add a new Caree

Add a new Caree to manage their condition via Asthma+me.

Add Caree

Completing Your Registration

To complete the Registration for your free **Activ8lives** Account, add the correct details for yourself by following the instructions on the App and selecting **Next** at the bottom of each page.

As you complete the information, your **Activ8lives** account begins to learn more about you and your health.

These data are used to help you with your self-care through self-monitoring. You can always edit these data later in **Settings** if you make an error, but please try to be accurate in entering your weight, your height, your age and gender. These are important data.

< Menu Registration ?

Current Weight & BMI

Your current BMI is 25.8. This is in the Overweight range.

66kg Weight

25.8 BMI

Weight Target

Your healthy weight range is between 47.4kg and 79.4kg

55kg Weight

21.7 BMI

Activity

What are your activity targets? Choose one of the defaults below, or enter your own.

Beginner

If you're new to activity tracking and wish to start gently

6,000 steps a day

35 minutes a day

Recommended

This is the recommended target for most people. It provides a reasonable daily challenge

10,000 steps a day

60 minutes a day

Advanced



This is an advanced target for people that are confident and experienced with activity tracking

12,000 steps a day

70 minutes a day

Step Target



10000 steps

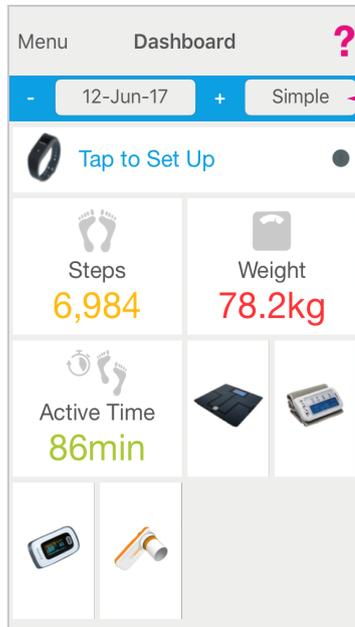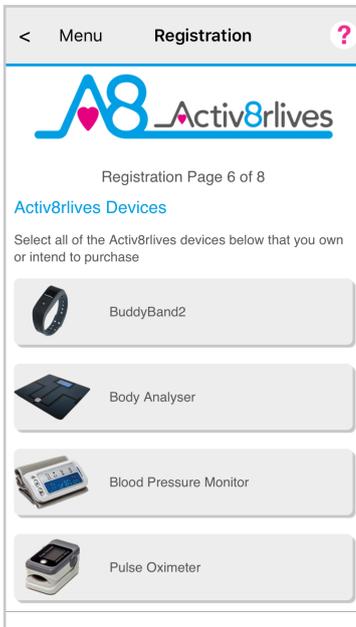

Completed Registration

During the **Registration** process you can select the **Activ8rives** devices that you have.

Once you have completed your **Registration** you will automatically be directed to your **Dashboard** and a summary of your activity and weight targets are set into this **Simple Dashboard**. These devices will now show up on your **Dashboard** so that you can **Quick Measure** by touching the image of the device and it will start the upload process.

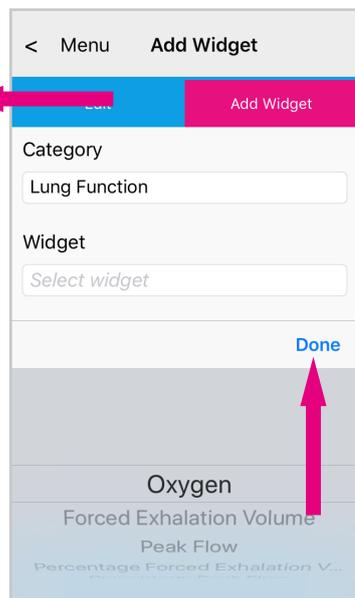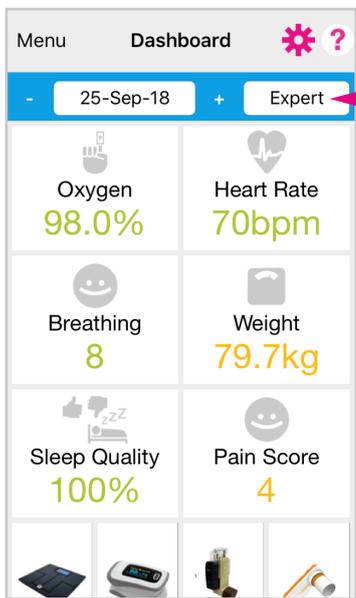



Dashboard



To add other health parameters to your **Dashboard**, such as Heart Rate or Pulse Oximetry readings, change and select either the **Advanced** or **Expert Dashboard**, these include increasing numbers of trackers. Touch the **Cog** in the top right-hand corner to add more **Widgets** to your **Dashboard**.

Help and Videos on App

Integrated into the **Activ8rives⁴ Health+Wellness App** is **Help** sections that can be reached from the

Menu or from any page which displays a **?** in the top right-hand corner of the screen. Scroll through this to access helpful information, **PDF Manual**, **Movies** and a **Quick Guide**. Touch the **!** symbol to reveal helpful hints throughout the **App**.

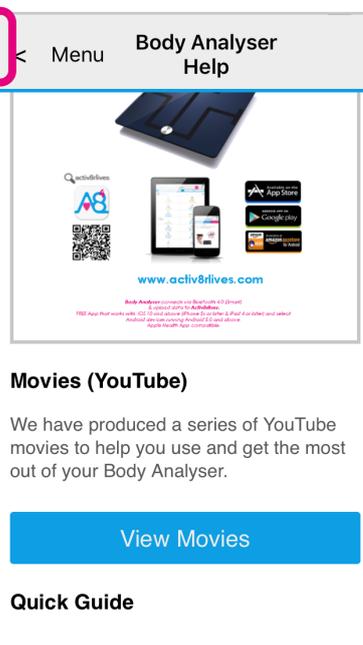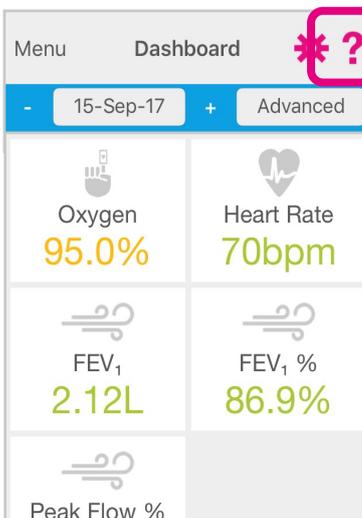

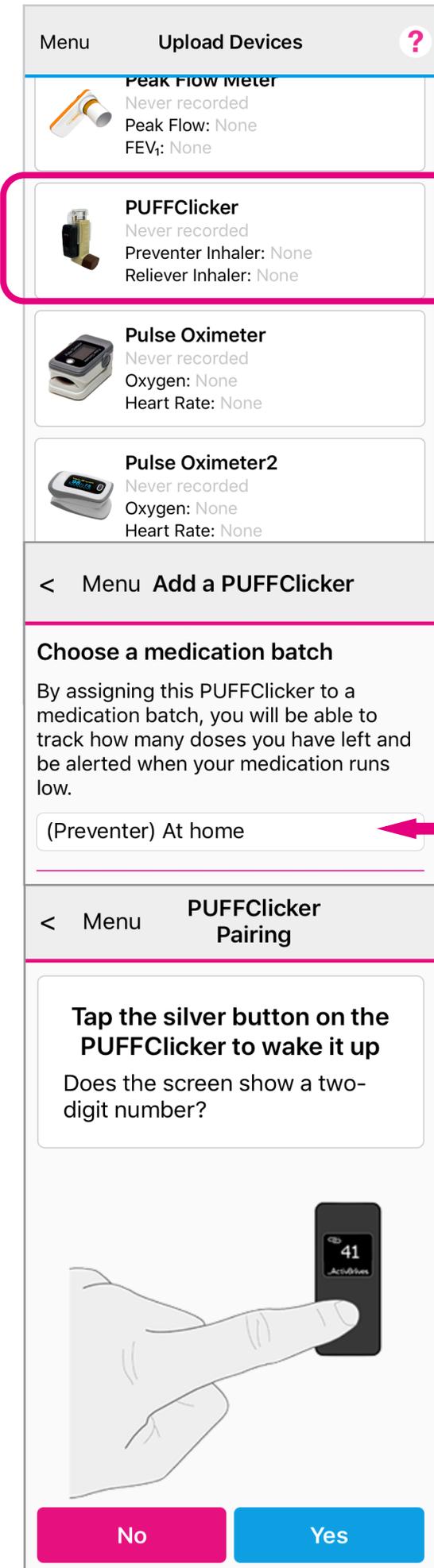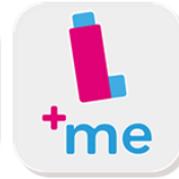

Pairing Your PUFFClicker

The **Activ8lives** PUFFClicker now needs to be paired with your Smartphone or Tablet. Ensure that your Smartphone or Tablet is connected to the internet with **Bluetooth switched on**.

Open the **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** on your Smartphone or Tablet.

Click Menu and then Upload Devices and touch the **Activ8lives** PUFFClicker, which then takes you through easy steps to choose which **Medication Batch** you want to connect. In this example, we have selected the (Preventer) At home. See [Page 18](#) for instructions to add your **Medication** before connecting your **Activ8lives** PUFFClicker.

Identifying Device by Pairing Code



The first time you set-up and try to connect your **Activ8lives** PUFFClicker to the **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** you will need to identify your device for added security, protecting your data and making it safer.

Wake Up PUFFClicker to Pair

So that your Smartphone or Tablet can see your **Activ8lives** PUFFClicker, wake it up by touching the **Silver Button**. If the screens look like the images below with a 2-digit Pairing Code and interlocked links then it is ready to pair. If your **Activ8lives** PUFFClicker screen does not display these screens follow the instructions on [Page 29](#).



Screen shows 2 rings and pairing code or Interlocked rings and pairing code

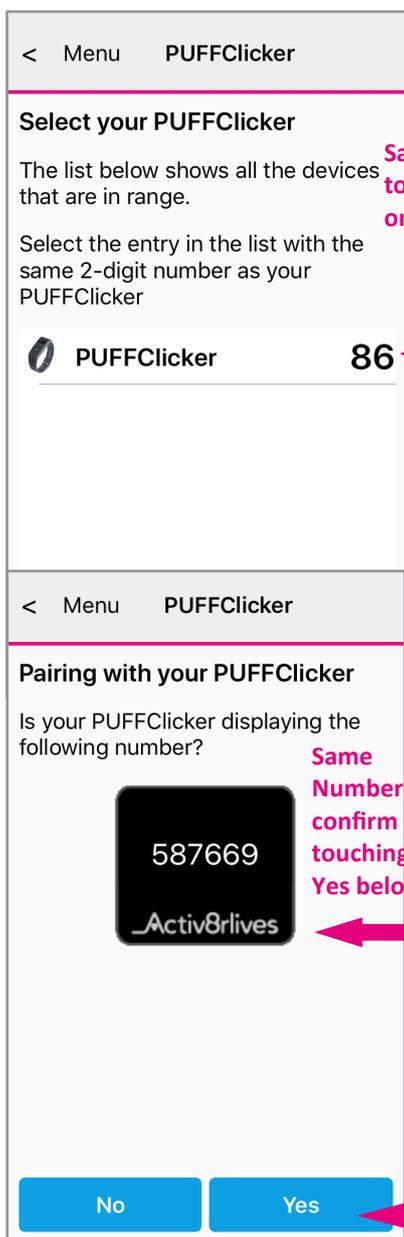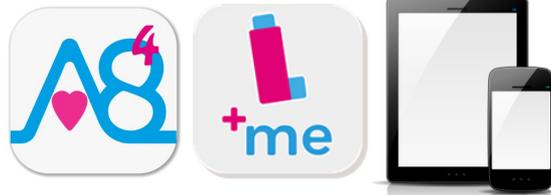

Same Number so touch the number on the App screen



Pairing Your PUFFClicker


The **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App** displays a 2-digit numerical code of the **Activ8lives PUFFClicker** that it finds within range of your Smartphone, and you may see more than one displayed here if others are in Bluetooth range.

If this code matches the number on your **Activ8lives PUFFClicker**, confirm by touching the number on the **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App**.

Second Secure Pairing

As a further step to ensure that your data is transferred to your Smartphone or Tablet safely, you are asked to confirm whether the unique 6-digit code on the **Activ8lives PUFFClicker** is the same as that on the **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App**.

If the two numbers are the same, confirm by touching **Yes** on the **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App** screen and the screen will then move to the **Inhaler Use Summary** page.

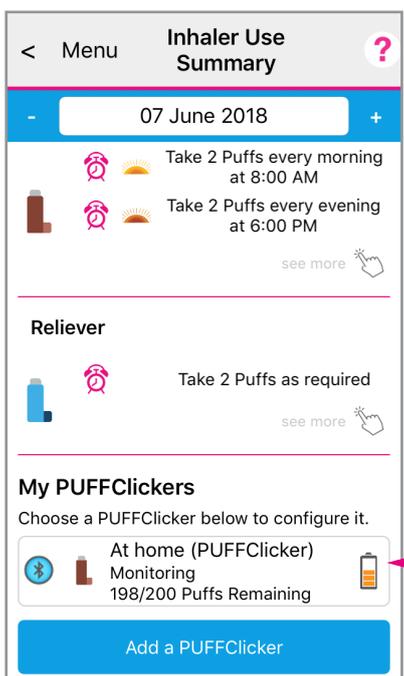

PUFFClicker Monitoring & Inhaler Use

The pairing is successful when the **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App** says **Monitoring** next to the image of the device.

The remainder of the screen gives a summary of your medication use for the date displayed.


Note: It also gives an example where the number of doses left 198/200 Puffs Remaining.

Note: Up to 4 **Activ8lives PUFFClickers** can connect to either the **Asthma⁺me App** or the **Activ8lives⁴ Health+Wellness App** at any one time.



Putting your PUFFClicker into Pairing Mode

If your **Activ8lives** PUFFClicker does not show a 2-digit **Pairing Code**, it needs to be put into the **Pairing Mode** and you will need to follow **Step 1** through to **Step 5** as shown below.

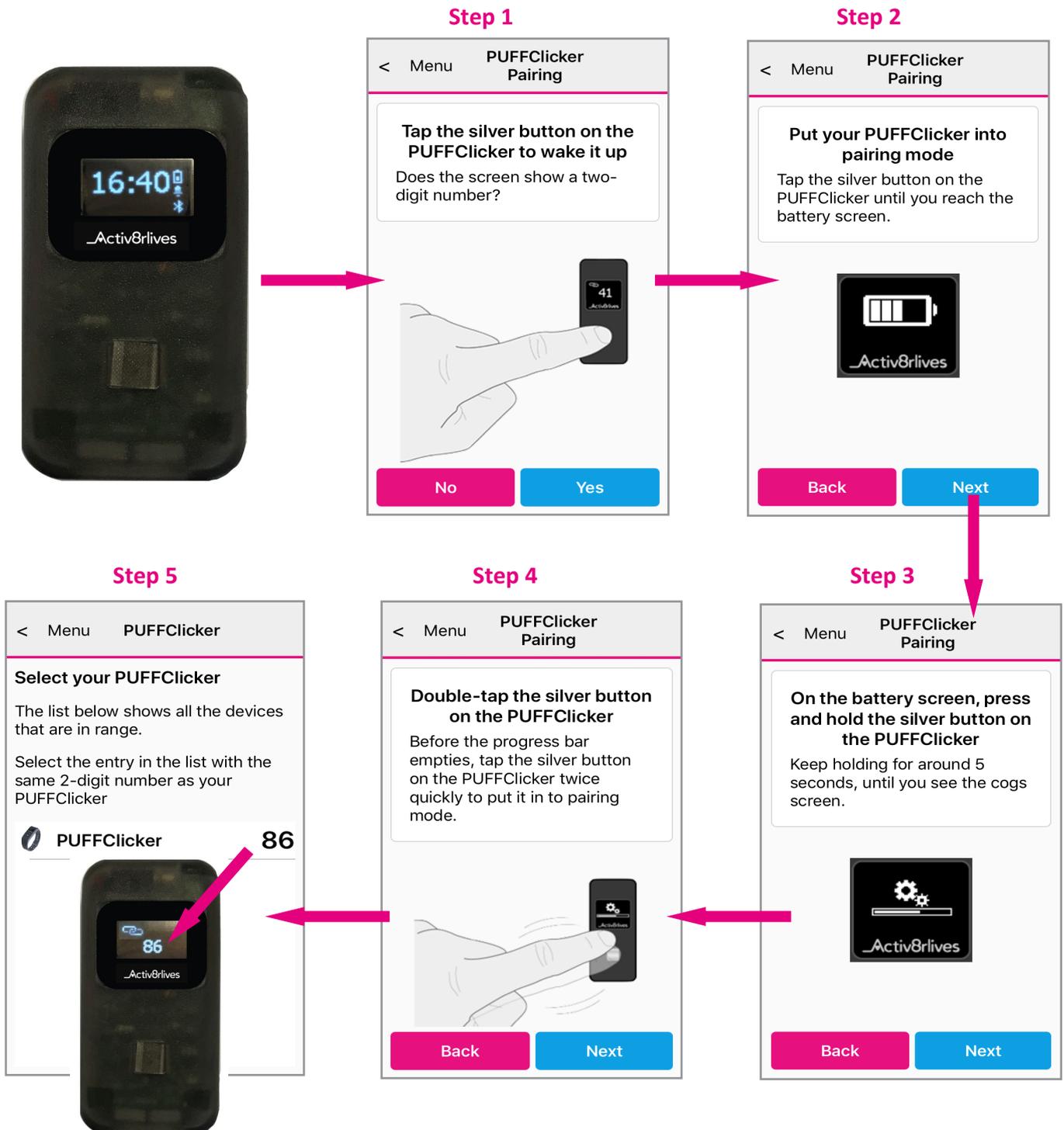




Navigating PUFFClicker User Interface Screens

Your **Activ8lives** PUFFClicker has 4 screens (6 screens if you have enabled step counting) and to move to the next screen touch the **Silver Button** once.




Pairing Screen indicates your unique 2-digit pairing code, changing between 2 rings and 2 interlocked rings.

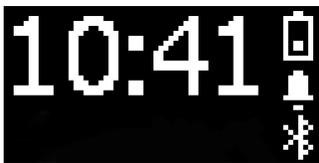


Time Screen (default screen) indicates the time in a 24 hour format, battery status, and Bluetooth connection.



Shaken Inhaler Screen indicates that you have shaken your inhaler before use.

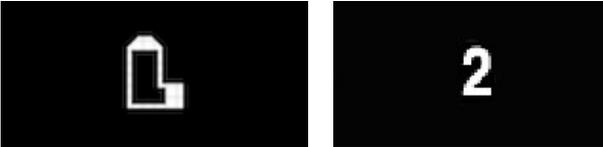


Inhaler Use Screen indicates the puff count you have taken.

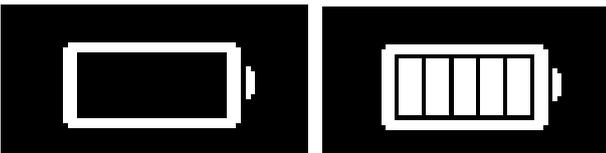

Battery indicates the energy remaining.



Today's Step Count and progress bar, if you have enabled Step Counting on your **Activ8lives** PUFFClicker.

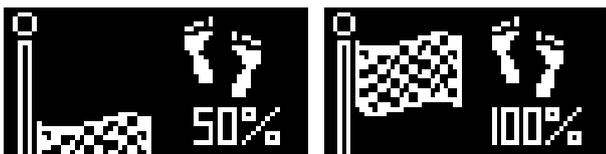

The **Activ8lives** PUFFClicker if you have enabled Step Counting will notify you with a vibration when you achieve 50% and then 100% of your personal Step Target. To change your daily Step Target select **Settings** from the **Menu** and scroll down to the **Step Target** to make changes.



Taking a Dose using your PUFFClicker

Different Inhalers need to be taken using different techniques to ensure they have maximum effect. Make sure to check the correct technique for the type of inhaler that you use by going to the **Lessons** (only available in the **Asthma+me App**).

If you use a pMD inhaler you should always use a spacer to make sure as much of the medication as possible gets into your lungs where it is most needed. You should **shake your inhaler before each dose**.

If you have an **Activ8rlives PUFFClicker** it is designed to buzz once if your inhaler has been shaken. **After each breathe, wait for 30 seconds** or until the progress bar on the **Activ8rlives PUFFClicker** has counted down after each puff before shaking for the next puff.



If you have already paired your **Activ8rlives PUFFClicker** to the **Asthma+me App** or the **Activ8rlives⁴ Health+Wellness App**, you can record a puff count, even when you do not have your Smartphone with you. The **Activ8rlives PUFFClicker** holds this in memory until it uploads when near your Smartphone or Tablet.



Steps for Child to take a Reading:

1. **Shake the Inhaler** until the **PUFFClicker** buzzes once and displays a tick.
2. Connect to the **Spacer**.
3. With Spacer mask (ensure there is a **tight seal** around the mouth and nose) or mouthpiece (ensure that the mouthpiece protrudes beyond the teeth and the lips are **tightly sealed** around the mouthpiece), press down on **Cap** of **Activ8rlives PUFFClicker** to dispense the first dose.
4. Breathe in: **Younger Child** using a face mask breathe 10 times and **wait 30 seconds** and **reshake inhaler** before next dose. **Older Child** using a mouthpiece breathe in 1 or 2 long deep breathes, or as described by your Asthma Nurse, **wait 30 seconds** and **reshake** before next dose.
5. **Reshake** the Inhaler 30 seconds after first dose in order to deliver a second or subsequent dose—repeat steps 1-4.
6. For the **Younger Child**, wipe their face with a damp cloth to remove any medication residue left on their skin and **ALL** children should either drink something or **brush their teeth after their dose to remove any medication residue**.



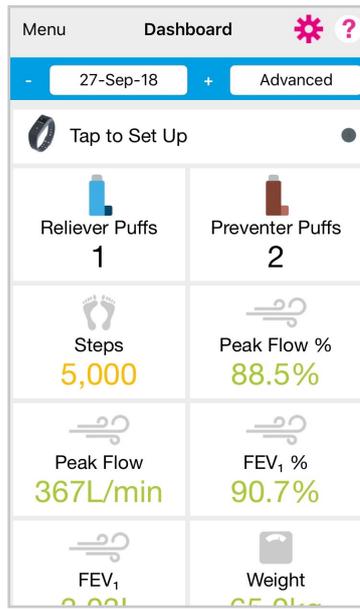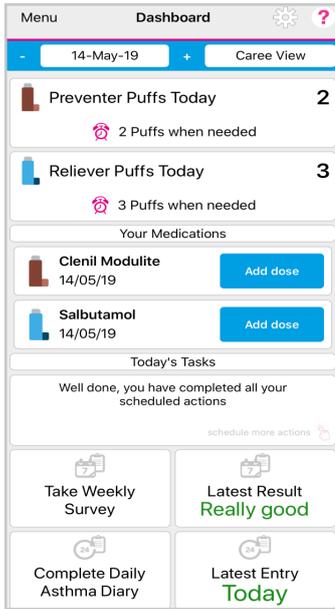


Viewing Your Data

Detailed data can be viewed from your **Dashboard** on either **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** by selecting **Dashboard** from the **Menu**. Touch a tile on the **Dashboard** to open specific data from a device.

When your **Activ8lives PUFFClicker** is connected via Bluetooth and WiFi/Cellular it will update during the day. These data will display on the **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** and will go back to zero at midnight.

View of **Dashboards** using the **Asthma+me App** (left) and **Activ8lives⁴ Health+Wellness App** (right).

If your **Activ8lives PUFFClicker** is **NOT** connected to your Smartphone or Tablet, the data will be stored on the device for up to 30 days but we recommend daily upload for motivation and in case you lose your **Activ8lives PUFFClicker**.

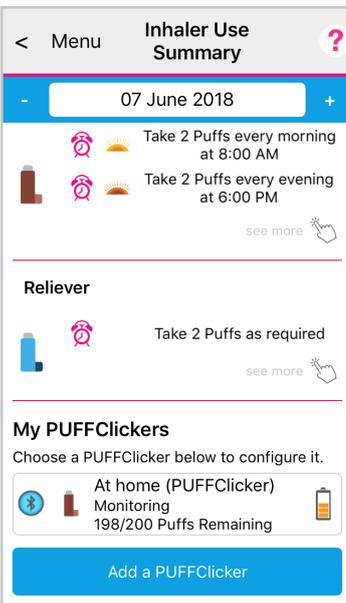

Your data can be shown in **Charts** by selecting it from the **Menu**. You can scroll down the page to see other parameters of your health, such as your medication use, lung function, weight and wellness levels.

You can display your data over different timeframes, by changing the time from the bottom **Range Days**: All, 7, 30, 90 or 365 days.

Colour coding of parameters within normal **Green**, alert **Amber** and out of parameter **Red** readings are easily seen on these **Charts** where the background colour indicates the population norms.

View of **Inhaler Use Summary** and **Charts** are the same using either the **Asthma+me App** (left) or **Activ8lives⁴ Health+Wellness App** (right).

A **Trackball** function allows you to touch a bar chart or data point for a specific day and it will give the reading and date. You can scroll left and right to look at your data.



Adding Data Manually



There may be a time when you want to add some health data to your account (Caree) or that of your Child.

This may include many other aspects of your health and wellbeing that may affect your Asthma or perhaps it does not affect Asthma but might be related to another health condition.


You can add this data manually to your **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** using the preconfigured categories listed.

From the **Menu** select, **Add Data** and then open up the types of data to be added.


Below is a Scenario where a Child has a raised body temperature and this may trigger Asthma symptoms.

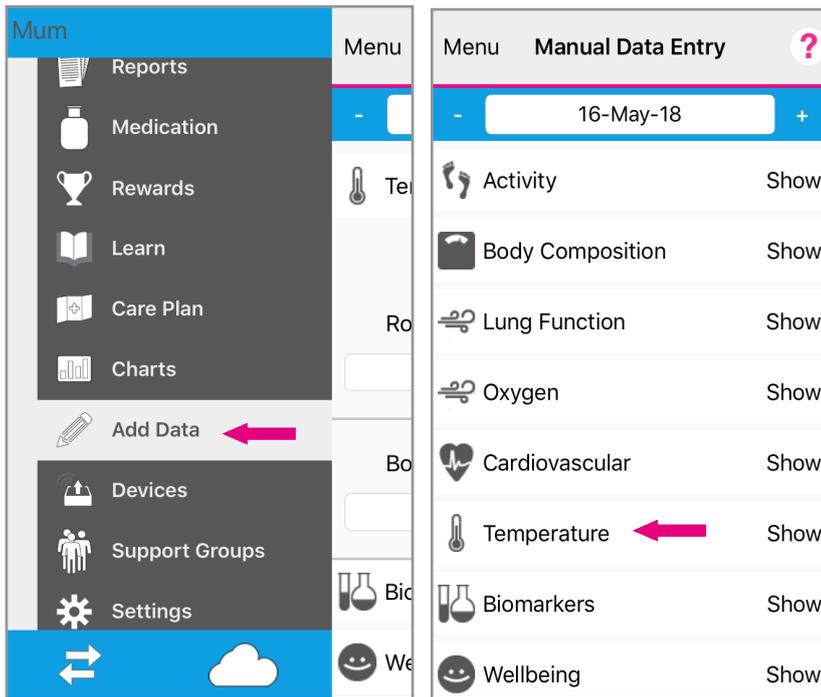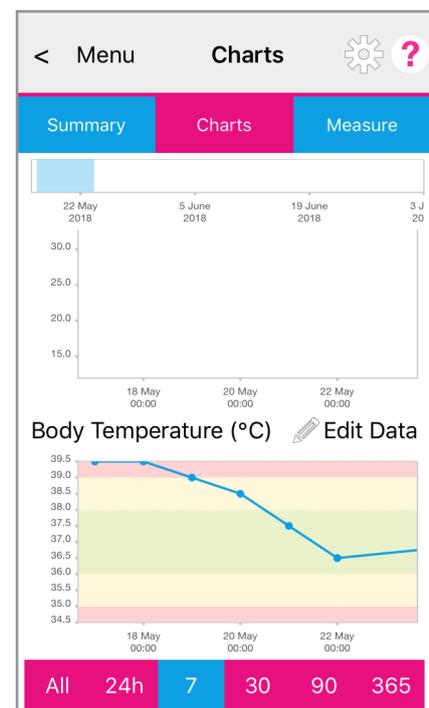

Activ8lives⁴ Health+Wellness App Activ8lives⁴ Health+Wellness App

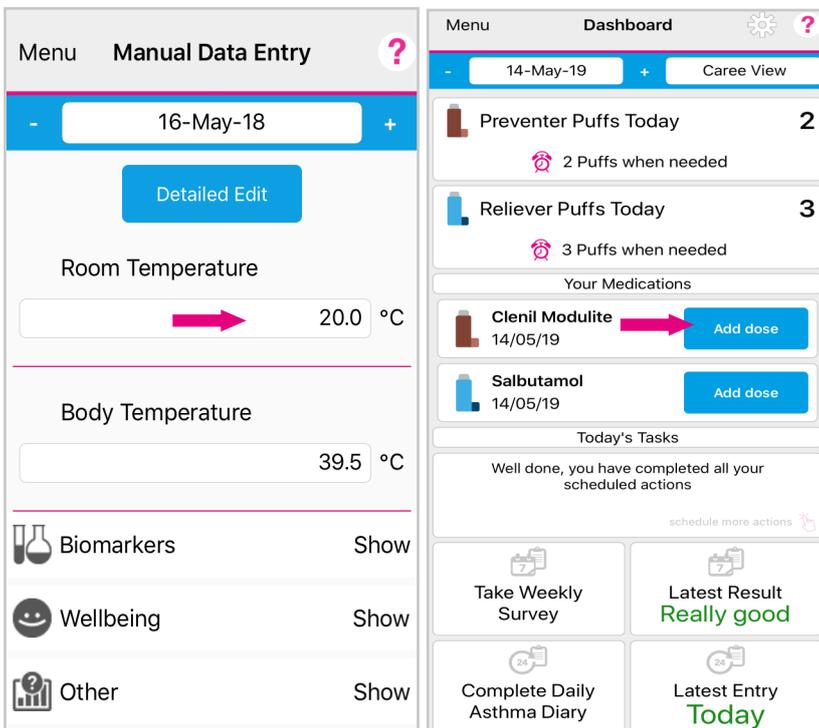

Activ8lives⁴ Health+Wellness App Asthma+me App

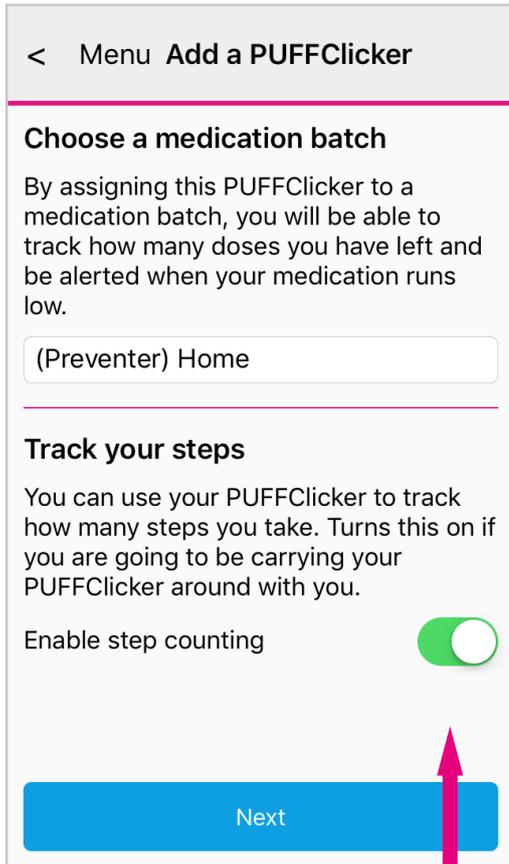

Enabling Step Counting on your PUFFClicker

If you would like to enable **Step Counting** on your **Activ8lives PUFFClicker**, navigate to the **Menu**, select **Devices** and scroll to the bottom to **Add a PUFFClicker**. Toggle on the **Enable step counting** and click **Next**.

Calculating Your Stride Length

Your **Activ8lives** account has some great things you can do with your data from the **Activ8lives PUFFClicker**. This makes it fun to use - especially if you are part of a group or team. We all tend to do the things we like doing. So having fun is part of the motivation for changing our lifestyles. Having fun, creating new challenges and goals, while being more active, also means that we enjoy our new healthy lives.

The **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** has a couple of great features, which we use to have fun: it automatically counts the distance we walk, the total activity time and estimates the calories we use during the day.

For the **Activ8lives PUFFClicker** to work these out for us, we need to enter our weight (kg, lbs or stones/lbs) and our stride length (cm or inches).

If you have never calculated your stride length before, here is how we do it.

Find a space in which you can measure 10m. This could be anywhere — indoors or outdoors — any place that you can walk without having to stop.


Make sure that you mark the beginning and end of the distance.


Then just walk along this distance. Try to use a normal stride and speed. Count the number of steps you take.

Now the mathematics! Divide 1,000cm (10m expressed as cm) by the number of steps you made.

For example, if you took 16 steps, your stride length is $1,000 \div 16 = 62.5\text{cm}$. (Most of us use a calculator to do this).

This is the value you enter into your **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** accounts for use with counting steps with the **Activ8lives PUFFClicker**.

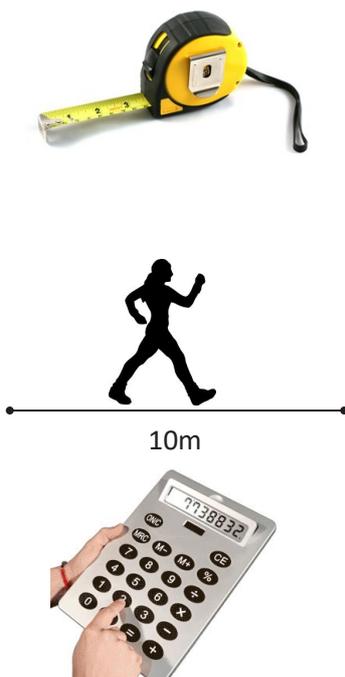

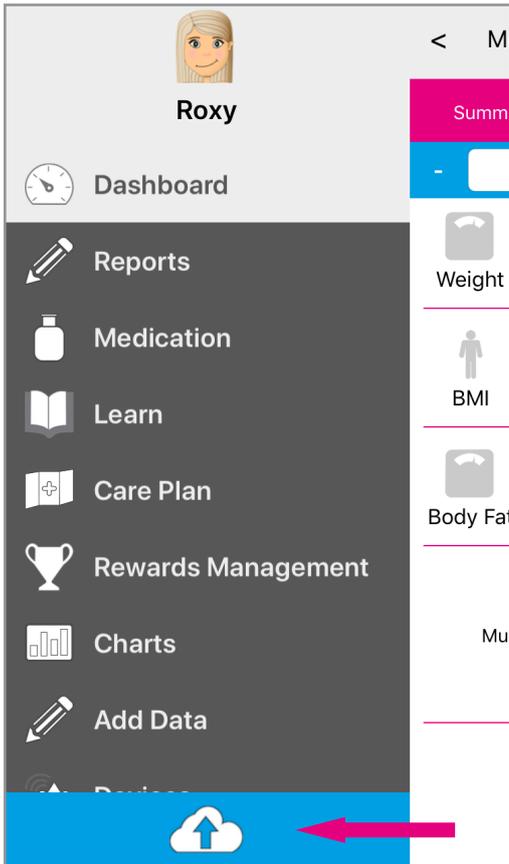

Sending Your Data to Cloud

If your Smartphone or Tablet is connected to the internet by WiFi, your data will upload to your online **Activ8lives** account automatically.


If there is an arrow in the **Cloud** it indicates that you have data to upload to the **Cloud** Server. Touch the **Cloud** icon and it will automatically detect and synchronise if you are on WiFi.

Touch the **Synchronise** bar to upload via Cellular network to upload the data.

By touching the **Download** bar your data will download from the **Cloud** for you to view on the App.

If there are no networks available the data will be uploaded at a later time by either WiFi or Cellular when you are within range. The data is not lost and you do not have to take the reading again.

When the data is stored in the **Activ8lives** secure **Cloud**, this means that you can view your data (from today, yesterday, a week, a month or years ago) at any time via the Smartphone App or **Activ8lives** website. www.activ8lives.com

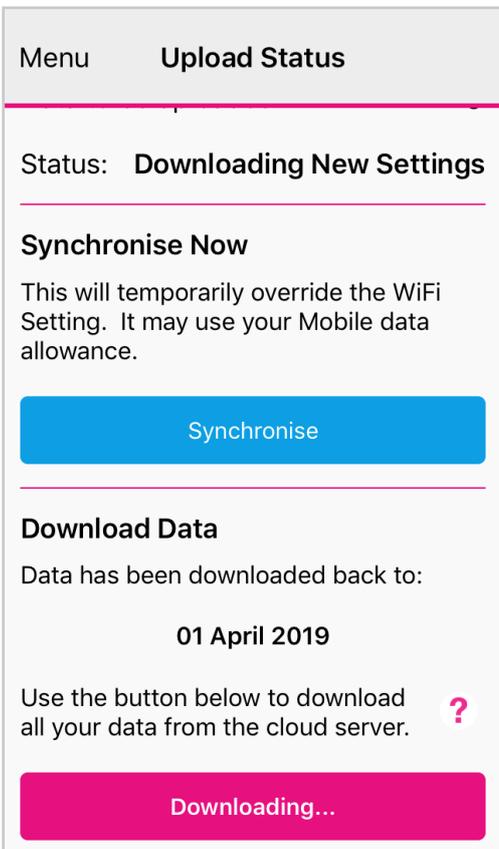

Data Security

The storage of your data is hosted on our private **Cloud** Server operated by UKCloud (Farnborough, UK). No data ever leaves the UK and we work to the NHS Data Security and Protection Toolkit and Cyber Essentials Plus.






Trouble Shooting

Help is Always Available

Help can be found on most pages of the **Asthma+me App** or the **Activ8lives⁴ Health+Wellness App** in the top right-hand corner. Touch the **?** to expand the page and scroll down through the **Help** information, including viewing the **Help Videos** and **PDF Manual**. Touch **!** to reveal helpful hints..

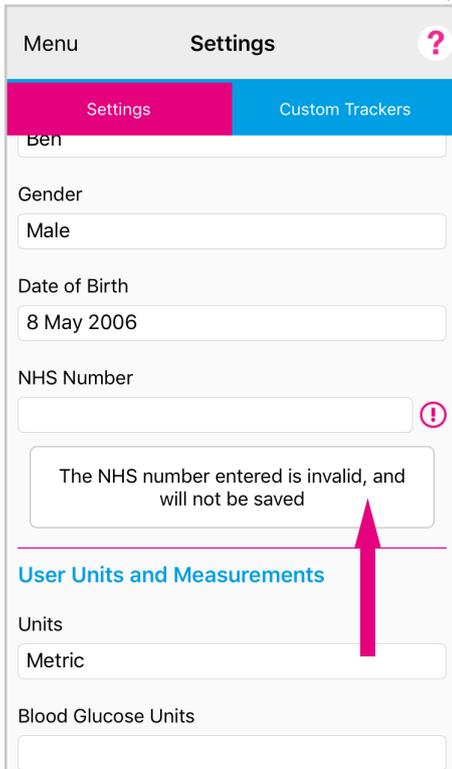

Will Not Connect?

Requires a Bluetooth Smart Ready device (typically Bluetooth 4.0 or later). iPhone 5 or later, iPad 4th Generation or later, iPad Mini 2 or later, all iPad Air and iPad Pro models or iPod Touch 6th generation. Android devices running 5.0 or later. Amazon Kindle Fire devices running Amazon Fire OS 5.0 or later.

If the **Activ8lives PUFFClicker** will not connect to Bluetooth, first check that **Bluetooth is switched on** your Smartphone or Tablet and then touch the **Silver Button** on the **Activ8lives PUFFClicker** to wake it up (**Page 29** provides help if your **Activ8lives PUFFClicker** needs to be put back into Pairing Mode).

App Crashes?

Close other Apps open on your Smartphone or Tablet, as these may be using too much memory to allow the **Asthma+me** or **Activ8lives⁴ Health+Wellness Apps** to function. Rebooting your Smartphone or Tablet often helps.

Not Recording?

There could be 3 reasons for the **Activ8lives PUFFClicker** not counting inhaler use:

- It may still be in **Pairing Mode** so you need to follow instruction on **Pages 27-28**.
- It may have lost Bluetooth connection and you will need to put it back into **Pairing Mode** and then reconnect to your Smartphone or Tablet on **Page 29**.
- Thirdly, your **Activ8lives PUFFClicker** may be stuck on **00:00** and it may need a **Reboot**. See **Page 37** for rebooting procedures.



Help Videos in **App**, follow this **?**



Website www.Activ8lives.com


YouTube <http://bit.ly/Activ8lives>

Still Need Help

We are here to help you 7-days-a-week, so please call us to help you set-up your device on UK +44 (0)1480 352821.



Place on
Charging Cable
Double Tap Silver
Button & Module
vibrates

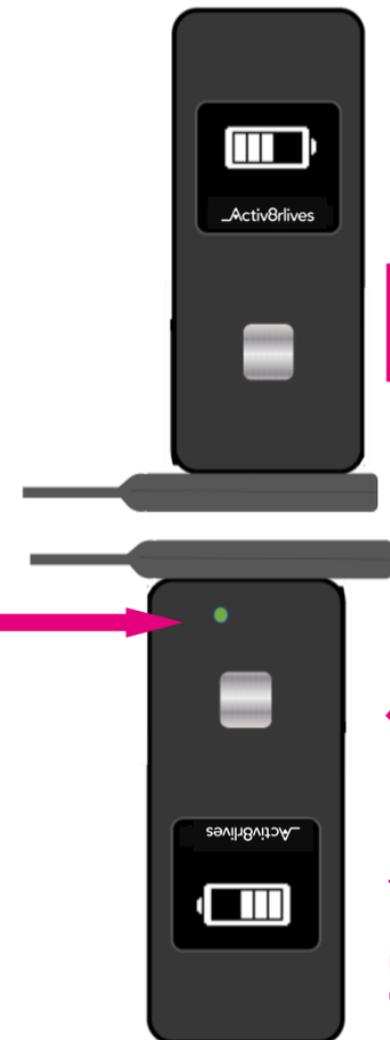

Rotate 180°
Touch pins together
Push against
magnets
Green light briefly shows

Reboots

Just as with many electronic devices, they sometimes need a reboot to take them back to the original or default settings and the [Activ8lives PUFFClicker](#) is no exception to this.

There are two methods for Rebooting:

Rebooting Without Data Loss

Your [Activ8lives PUFFClicker](#) can simply be rebooted by placing it on the **USB Charging Cable** as if you were charging it. Tap the **Silver Button** twice in quick succession.


The [Activ8lives PUFFClicker](#) will vibrate and your data should still be on your [Activ8lives PUFFClicker](#) after the reboot.

Rebooting With Data Loss

The first method of rebooting above is very effective but it may need the second method of reboot. **Note:** You may lose data unless you have already uploaded the data to your **Account**.

Rotate the [Activ8lives PUFFClicker](#) by 180° or half a circle. The metal magnetic pins on the bottom of the [Activ8lives PUFFClicker](#) module will try to push it away as you try to push together the metal magnetic pins on the **USB Charging Cable**.

TIP: Hold the **USB Charging Cable** in the same place with one hand. With the other hand take the [Activ8lives PUFFClicker](#) off the **Charging Cable** and rotate it so that the charging pins are pushing against the charging pins on the [Activ8lives PUFFClicker](#). You will have to juggle the module and **Charging Cable** to push the pins together until you see a **Green LCD light** come on within the [Activ8lives PUFFClicker](#) module. This will indicate that it has been rebooted.

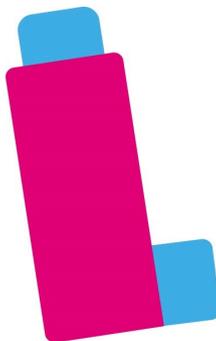

PUFFClickerTM

Smart pMD Inhaler Tracker

Specifications

Type	Description
Dimension (L x W x H)	85mm x 45mm x 40mm
Weight of Module	25g (including battery)
Weight of PUFFClicker & Inhaler	35g
Inhaler Shaking	Conformational tick
Doses Counted	0-99,999 doses
Steps taken	0—99,999 steps
Memory	30 days on PUFFClicker
Stride distance input range	10—150cm
Age input range	5—90+ years
Communication Interface	Bluetooth Smart (4.0 or BLE)
Units	Metric or Imperial
Clock	24-hour clock
Splash proof	Splash proof and wipeable only
Operating temperature	0—40°C
Storage temperature	0—50°C
Eco-friendly	Auto turn-off without operation
Packaging	Fully recyclable
Rechargeable Battery	3.7V 100mAh rechargeable lithium ion battery
Magnetic USB Charging Cable	DC 0.5—1 Amp charger. 120 minute charge time, lasts 5-7 days with typical use.

Labels and Symbols



This product is manufactured by Aseptika Limited (Activ8rlives), Suite 5, LDH House, St Ives Business Park, Parsons Green, St Ives, Cambridgeshire, PE27 4AA, UK.

Model [Activ8rlives PUFFClicker™](#).



Battery Integrated Lithium Ion battery. Input—DC—100mAh.



Made in compliance with 93/42 EEC Directive on class 1 Medical Devices, amended by 2007/47/EEC.



RoHS Directive (2011/65/EU).



The symbol in accordance with IEC EN 60601-1-2: 2007 in section 5.1.1 for products including RF transmitter.



In accordance with IEC 60601-1 the product and its applied parts re type BF and thus protected against the risks of electrical leakage.



This symbol is required by European directive 2002/96/EEC on waste electrical and electronic equipment (WEEE). At the end of its useful life this device must not be disposed of as normal domestic waste. Instead it must be delivered to a WEEE authorised collection centre. Due to the construction materials used for the device, disposal as normal waste could cause harm to the environment and/or health.



Read instructions before use.

Warranty

[Activ8rlives](#) (Aseptika Limited) guarantees this product for one (1) year after the date of purchase. The guarantee does not cover the battery, packaging and damages of any kind due to misuse, such as: dropping or physical misuse caused by the user or water damage outside of the stipulated usage. Claimed products will be replaced when returned marked as **DAMAGED** together with the original proof-of-purchase. For any general enquiries or technical questions concerning the product or questions about the warranty, please contact [Activ8rlives](#) — details are on the back of this User Manual.



Aseptika Limited (Activ8rlives)

Suite 5, LDH House

St Ives Business Park

Parsons Green

St Ives

Cambridgeshire

PE27 4AA

United Kingdom

t: +44 (0)1480 352 821

e: support@aseptika.com

w: www.activ8rlives.com

Copyright © 2019. All rights reserved.

ASL TF-003 MAN010 PUFFClicker V2.0